



Diagnos COVID Rapid Antigen Test Instruction

1. Remove the test device from the packaging. Put it on the table to standby.
2. Twist off the tab of the Antigen Buffer Solution (sealed in the plastic sterile single-use container), and transfer the Antigen Buffer Solution to the plastic container with the purple cap.
3. Close the purple cap. Put the container with the Antigen Buffer Solution on the table to standby.
4. Remove the swab from the packaging (take it out from the “open” area). Hold the plastic part, and do not touch the cotton swab part.
5. Tilt your head back. Gently insert the swab (the cotton part) about 1 inch into one of your nostrils. Twist for 5 seconds. Repeat the 5 seconds twist in another nostril.
6. Remove the purple cap of the container with Antigen Buffer Solution. Insert the swab into the container, and stir the swab into the buffer solution by twisting the swab (the cotton part) back and forth 20 times. Please check that the swab of the cotton tip is fully submerged in the solution.
7. Close the purple cap tightly.
8. Twist off the top half of the purple cap to expose the dropper tip.
9. Hold the container vertically. Squeeze and drip 3 drops of the solution into the circular opening of the test device (indicated by “S”).
10. Please wait for 15 minutes and read the result.
11. If you see one solid line in “C” area, that is negative result.
12. If you see one solid line in “C” area, and one solid line in “T” area (seeing two lines), that is positive result. Please contact us, or Department of Health immediately. We will assist you to arrange the COVID-19 RT-PCR test in the local testing institutions recognised by the Hong Kong SAR Government.

If you would like to learn more about the testing and sampling procedure, please scan the QR code above (the testing and sampling demo video is uploaded).

If you have any question, please feel free to contact us at 35758473, or WhatsApp at 67493545.

Thank you for your attention.

